



## **Bioclear Magic Mix Pre-Polish Paste Instructions For Use (IFU)**

### **Indications:**

The Bioclear Magic Mix Pre-Polish Paste is indicated for all anterior and posterior restorations where polishing of teeth is needed.

### **Intended Purpose:**

Magic Mix Pre-Polish is used for reducing fine grooves and lines on the surface of composite restorations in advance of the final polish. Magic Mix Pre-Polish is invasive through a body orifice, specifically orally, but do not penetrate past the pharynx.

### **Safety:**

To offer your patients the safest, most comfortable procedure possible, carefully read the enclosed instructions and watch the tutorials on the website prior to use.

Note: Please check [www.bioclearmatrix.com](http://www.bioclearmatrix.com) to find the most recently updated instructions and tutorials.

### **Precautions:**

Bioclear Matrix offers instructions and classes for using these products. Any use not specified in the instructions is not recommended and will be the responsibility of the attending practitioner.

### **Instructions:**

- 1) Put some of the Magic Mix in a disposable cup
- 2) Use briefly and then go back to get more mix on the cup every 3 or 4 seconds. Repeat
  - \*Note: Make sure to keep the cup and teeth wet so it does not overheat the composite.
- 3) Magic Mix is water soluble, so when you are finished, you can wash away the remaining mix with water.


When finished with the Magic Mix, there should be omnidirectional scratches and a slightly matte finish that will then be further finished with a polisher.

### **Composition:**

See Magic Mix Safety Data Sheet (SDS). It can be found online at

### **Limitation of Liability**

Magic Mix is intended to be used by educated dental professionals, e.g., general dentists, dental assistants, dental hygienists, who have theoretical and practical knowledge on usage of dental products only. Any use not specified in the instructions is not recommended and will be the responsibility of the attending practitioner.

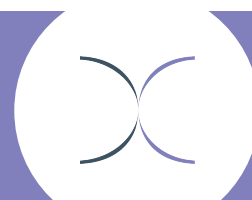

**Manufacturer:**

Bioclear Matrix Systems by Dr. David Clark  
3402 South 38<sup>th</sup>  
Tacoma, WA 98409  
USA  
Ph: 855-712-5327

**Origin:** Made in the USA

Any serious incident that has occurred in relation to the use of this device should be reported to the manufacturer immediately either by mail at the address above or by email to [contactus@bioclearmatrix.com](mailto:contactus@bioclearmatrix.com).

**Importer:**

Find the importer for your country on our website at [www.bioclearmatrix.com](http://www.bioclearmatrix.com)



**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

**IFU Revision Date:**

May 4, 2021

